



Spenco RX® ThinSole™ Orthotic Arch Supports Full and 3/4 Length

Full Cat. No. 43-307
3/4 Cat. No. 43-240



Target Consumer:

Consumers with flat, low, or fallen arches.
Consumers seeking a thinner solution for firm arch support to control foot position.

Recommended For:

Treatment of arch pain. May be molded for a custom fit to support high arches.

To help reduce pronation and supination for improved stability.

Choose ¾ length for additional toe room in tighter fitting shoes.



Best For Use In:

Dress and casual shoes (or boots).



Components*:

Composite, Spenco® Material, Fabric, Arch Height: 18.32mm (3/4in), Thickness at Heel: 4.48mm (3/16in), Thickness at Forefoot: 2.90mm (1/8in)



Instructions:

Trim forefoot area with scissors if needed. To adjust the arch height, see your local footcare professional. LAB DIRECTIONS: Using a heat gun at 300° F heat the bottom surface until the support is pliable. Adjust immediately. Maintain the adjusted position until cool. Repeat if needed. (Use heat protective gloves.)



Care:

Hand wash in warm water. Air dry.

Sizing Chart:

Size	0	1	2	3	4	5	6
U.S. Women's	3/4	5/6	7/8	9/10	11/12		
U.S. Men's			6/7	8/9	10/11	12/13	14/15
U.K.	2/3	4/5	5/6	7/8	9/10	11/12	13/14
European	32/34	36/38	38/40	40/42	42/44	44/46	46/48

Features & Benefits:

- 1 Full Length** provides complete heel-to-toe comfort.
- 2 Air injected Spenco® Material** Exclusive cushioning layer absorbs shock and impact to provide maximum comfort.
- 3 Thin Orthotic Shell** provides excellent stability without the added bulk of most orthotics.
- 4 Firm Adjustable Arch Support** allows for greater arch support when heated and molded to individual preferences.
- 5 7 Sizes** for correct arch placement for better overall fit, comfort and performance.
- 6 4-way Stretch Fabric with Ultra-Fresh Silpure Antimicrobial** helps prevent blisters while controlling odor.

One-Year Unconditional Guarantee

Call toll-free: 1-800-877-3626
In Canada: 1-800-387-9538



www.spenco.com



*Measurements are approximate and vary by size.